# EliRamp®

## NO

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## GB

EliRamp is delivered in seven standard modules that cover all threshold heights from 1.5 to 12.5 cm with 0.5 cm intervals. All the modules, except for the smallest one, has one or more vertical grooves in the top (picture 1). The grooves are intended to be cut along using a Stanley knife, saw or jigsaw (picture 2). Height indicators are embossed into the ramp as numbers and arrows. The top one indicates total height, the others the height of each groove. Due to EliRamp's patented design, the rubber in the grooves that the material is cut along is no more than 2 cm thick, regardless of which module you are cutting. Each groove cut reduces the height of the ramp by 0.5 cm and its depth by 4 cm. The incline remains unchanged at 1:8, or 7.125°. The ramp can be easily trimmed with a saw/jigsaw (picture 3). Clean the area before installing the ramp. Simply put the EliRamp in place, or attach it to the floor (indoors) using double-sided carpet tape (picture 4). The ramp can also be secured both indoors and outdoors using a one-component construction adhesive (picture 5). Remember to degrease the rubber with hot water and washing-up liquid when using tape or adhesive.

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